
# FLEX™

## Instructions for Use Mode d'emploi Instrucciones de uso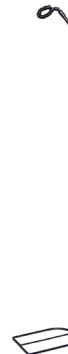

### To Assemble:

1. Remove the FLEX CPAP Tubing Hanger from its pouch.
2. Unfold each section of the FLEX and connect each adjoining piece so that all pieces are secure when completed.
3. Insert the base of the FLEX between your mattress and box spring or bedframe at the desired area of your bed.
4. Insert your CPAP hose through the coiled loop at the top of the FLEX arm. To adjust the tension of the loop, hold the plastic toggle and then pull gently on the lower end of the coil.

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**JACK™**

- 1 Remove protective film from suction cup. Do not touch the adhesive surface with your fingers in order to promote stability of the product. Bend all edges of the suction cup. Should be washed if repositioned.

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**JACK™**

If Jack does not stick properly anymore, follow these steps:

1. Wash the suction cup in water with mild liquid soap and rinse under running water.
2. Leave the suction cup to air dry or use a hair dryer.
3. Once completely dry, Jack is now ready to be used again!

The support provided applies to suction cups installed under ideal conditions and should be used in a responsible and reasonable way. The suction capacity varies from surface to surface. KEGO Corporation is not responsible for any damage resulting from use or misuse of the product. Do not overload or over-pull. Do not use as a grab bar.

**Caution: This product is for indoor and temporary use only. Packaging and small parts can be dangerous. To avoid danger of suffocation, keep this product away from babies and children.**

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