Disinfects CPAP Masks and Accessories,
and other Household items

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KEGO

3B[™] Medical, Inc.

Thank you for your recent purchase of the Lumin® disinfecting unit. Once you have cleaned your CPAP mask and accessories, or other household items, you can now disinfect them with the innovative Lumin® device. Using a very specialized and high powered UVC light source, the Lumin® can disinfect your items in only 5 minutes, offering a cleaner, safer, faster, and better choice. The Lumin® is intended to supplement your CPAP accessory manufacturers cleaning recommendations.

Please read and follow all directions in this user manual. Any questions can be directed to **support@kegocorp.com**.

Helpful Hint:

We recommend cleaning and disinfecting CPAP masks and accessories, along with any other personal care items twice weekly, and daily during cold and flu season.

Indications of Use

The Lumin® is a device intended to disinfect Continuous Positive Airway Pressure (CPAP) accessories, including mask and water chamber, and other household items. A person using a PAP device to treat Obstructive Sleep Apnea (OSA) may use the Lumin® to disinfect their mask and water chamber to augment their PAP equipment manufacturer's recommended cleaning instructions. The Lumin® is intended for home use. The Lumin® may also be useful as a multi-purpose disinfecting device for other personal care products (i.e. electric toothbrush, toys, cell phones, etc).

-  **Warning** - the Lumin® is not a toy and is not intended to be used by children.
-  **Warning** - the Lumin® has a powerful UV light bulb included, use with care in case of broken glass.
-  **Warning** - do not use in or around sinks, tubs, pools or any bodies of water.
-  **Warning** - the UV that is emitted in the closed chamber is not compatible with life, never place living creatures in the Lumin®.
-  **Warning** - do not stack objects on top of your Lumin®.
-  **Warning** - use this device only for its intended use as described in this manual. If the equipment is not used in the manner specified, the protection provided by the equipment may be impaired.
-  **Warning** - Do not attempt to operate this device with the drawer open. The device was designed to shut off automatically when the drawer is open. It is important not to defeat or tamper with this safety feature as UVC light can cause damage to eyesight.
-  **Warning** - The Lumin® plugs into an outlet. Disconnect the device by unplugging. Do not plug the device into an outlet that is difficult to access.

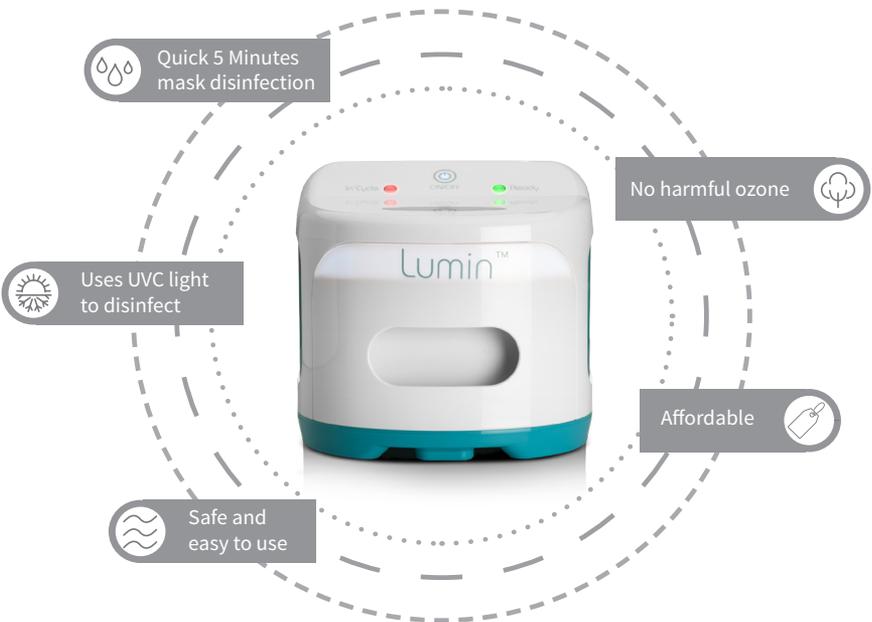
Important information concerning your safety or the operation of your device.

SPECIFICATIONS

SKU:	LM3000
Weight:	5.5 lbs
Dimensions:	12.25" x 8.5" x 7.75"
Voltage:	AC 110 V 60Hz AC 220 V 50Hz
Power Rating:	13W
Environmental Conditions:	Storage Temperature: 0° to 120° F / -18° to 49° C Operating Temperature: 20° to 100° F / 7° to 38° C Humidity: Up to 93%, non-condensing

FEATURES

Your healthy choice in CPAP accessory disinfection with simple one-touch operation.

Understanding Your Lumin®

The Lumin® operates using a specialized high powered UVC bulb. The drawer is made of polished aluminum which reflects the UVC light to improve coverage. The Lumin® provides disinfection of your CPAP mask, hose, humidifier water chamber, and other household items. Daily use of the Lumin® will help keep you healthy and compliant with your CPAP equipment.

Because the Lumin® operates using a form of high energy light, it is very effective in disinfecting your mask, hose, water chamber, and accessories. Disassembly of your mask is not required for daily routine disinfection. Once a week, however, most manufacturers recommend that you disassemble your mask to wipe down residue. A disassembled mask can be placed in the Lumin® for an even more thorough disinfection. Hoses do benefit from Lumin® disinfection, and the minimum recommendation of twice weekly will prevent build up of biofilm in the hard to reach areas.

Helpful Hint:

The Lumin® can disinfect a variety of other products safely, including dentures, hearing aids, and toothbrushes!

Using Your Lumin® Rack

Use of the clear Lumin® RACK is suggested to improve disinfection. Set your mask and accessories on the RACK to allow greater penetration of the underside of the article being disinfected. If your item is large, use of the Lumin® RACK is optional and it may be removed.

Using Your Lumin®

1. Plug the Lumin® power cord in a wall outlet.
2. For your humidifier water chamber, open the water chamber and lay both sides facing up. For your CPAP mask lay the mask with the inside facing up and laying flat for maximum UV penetration
3. If your mask or accessories are visibly soiled please first wipe thoroughly with a damp cloth or a CPAP wipe.
4. Close the Lumin® chamber drawer
5. Press round power button, the Cycle-On red light will turn on indicating that the 5 minute disinfection cycle has started.
6. A light glow of UV Light will be visible through the tinted oval window on the top of the unit while disinfection is occurring
7. After 5 minutes the Lumin® will emit a beep and the UV light will no longer be visible. A green light will indicate completion of the disinfecting cycle. You may now remove your equipment. No waiting period is necessary and your mask and equipment are now ready to use.

Caring for the Lumin®

The Lumin® bulb is designed to exceed the life of your device so changing the bulb should not be necessary. If, however, it does become necessary, the bulb is removed by simply grabbing the base and pulling firmly. The bulb is easily replaced by inserting the new bulb and pushing it into the socket. If you are in need of a replacement bulb please contact your local home care dealer.

The Lumin® drawer is lined with polished aluminum. Do not use baking soda or other alkali-based cleaners on aluminum as it can cause discoloration. Aluminum scratches fairly easily, so avoid abrasive cleaners and scrubbers when cleaning and polishing. If using a metal or glass cleaner, test the cleaner on a small portion of the drawer. Polish with a soft lint free cloth.

Should the casing of your Lumin® become soiled you may wipe down with any moist cloth. Do not immerse your Lumin® into water or any cleaning solvents. Place your Lumin® on a level counter or tabletop. Do not drop your Lumin® or the bulb and or glass drawer may shatter.

Disinfect your CPAP equipment in just **5 minutes!**

Open the Lumin®
sliding tray

Place your CPAP mask,
hose or humidifier
chamber on the
Lumin® tray

Close & push the
button to start

Frequently Asked Questions

How does Lumin® work?

Lumin® works by emitting high energy light within a narrow spectrum referred to as UV-C. The Lumin® relies on a low-pressure, mercury- arc lamp designed to produce the highest amounts of UV energy – where 90% of energy is generated around 254nm. The dose of UV-C emitted in one 5 minute cycle is sufficient to disinfect your CPAP mask and other household items. The use of UV-C is environmentally friendly, leaves no residue or toxic gases or chemicals.

Why doesn't the Lumin® use Ozone, like other CPAP cleaners?

Because the Lumin® was developed by 3B Medical, a specialist in CPAP equipment, masks, and accessories, they have a heightened level of commitment and concern to their patients. 3B Medical did not feel that the safety concerns of ozone being used in the home were being adequately disclosed. Due to the lifestyles of many patients, they also had concerns that the two hour wait period in using CPAP equipment might be ignored and harmful OZONE inhaled. The U.S. EPA published a serious warning about use of ozone in a home environment:

“When inhaled, ozone can damage the lungs. Relatively low amounts can cause chest pain, coughing, shortness of breath and throat irritation. Ozone may also worsen chronic respiratory diseases such as asthma and compromise the ability of the body to fight respiratory infections.”

<https://www.epa.gov/indoor-air-quality-iaq/ozone-generators-are-sold-air-cleaners>

For that reason, in designing the Lumin®, care was taken to find the safest and fastest way to disinfect CPAP accessories. UV-C was by far the best solution.

Why should I clean and disinfect my mask, hose, and water chamber?

Moisture left in your mask, hose and water chamber becomes a breeding ground for bacteria, germs and mold. Keeping your CPAP mask, hose, and water chamber free of harmful germs and bacteria helps avoid getting sick from a dirty CPAP. Clean your CPAP equipment prior to using Lumin®.

Are there any supplies that need to be reordered?

No. The Lumin® requires no replacement supplies whatsoever. The Lumin® relies on a special UV-C bulb that is rated to easily outlast the life of the device and should not require changing. In the event the bulb is damaged or does require replacement, low cost replacement bulbs are available at your local home care dealer.

How do I use the Lumin®?

The Lumin® is a single button device. It really could not be any easier. First wipe the inside of your mask and cushion with a CPAP wipe to remove any physical residue. If disinfecting your humidifier water chamber, open the chamber and place in the drawer face up. If disinfecting your mask, lay your mask in the drawer face up. Close the drawer and press the button. In 5 minutes the cycle is complete and your mask and water chamber are ready for use. Because UV-C disinfection involves light waves, it operates in “line of sight” fashion, only irradiating surfaces within its sight lines. If there is heightened need for disinfection, the mask can be disassembled or a second cycle used after flipping the object over. For general daily use, only one cycle is necessary.

Does Lumin® cause an odor?

UVC light has no odor. The vast majority of Lumin® customers experience no odor. If there is an odor it is likely similar to the smell of burnt hair. Household dust is 90% composed of dead skin cells. Skin, like hair, contains a sulfide molecule. UVC light kills pathogens by destroying DNA with high energy light. That process also frees the sulfide molecule in a skin or hair molecule. If your unit has an odor please wash thoroughly with a damp cloth using Dawn liquid detergent and dry with a paper towel. Run one cycle with the drawer empty and let the device sit overnight. Next day, your unit should have no odor. If you have any questions or comments, please feel free to email us at support@kegocorp.com.

WARRANTY

Your Lumin® is warranted for a period of 2 years from time of purchase to be free from manufacturer defect or workmanship. Should you encounter a warranty issue please contact your home care dealer where you purchased the device.

It is at the discretion of KEGO Corporation to repair, replace or exchange your Lumin® unit should it be necessary. KEGO is not liable for misuse, mishandling or breach of this written guarantee. Attempts to alter the Lumin® in any way will void warranty.

- ⚠ This equipment has been tested and found to comply with the limits for a Consumer ISM equipment, pursuant to Part 18 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.
- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- ⚠ Please note that changes or modifications of this product is not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.
- ⚠ This product may cause interference to radio equipment and should not be installed near maritime safety communications equipment, ships at sea or other critical navigation or communications equipment operating between 0.45-30 MHz.
- ⚠ If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

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**3B[®] Medical, Inc.**