Stop Snoring. I've got your back.<sup>TM</sup>  
Arrêter de ronfler. Je te couvre.<sup>MC</sup>  
¡Dejar de roncar. Tengo su espalda.<sup>TM</sup>

**Instruction Manual**  
**Manuel d'instructions**  
**Manual de instrucciones**

---

# Sleep Noodle™ Instruction Manual / Manuel d'instructions / Manual de instrucciones

**EN** Thank you for purchasing the Sleep Noodle™ by CPAPology™. This is the first step in enjoying a good night's sleep, for both you and your sleep partner.

Follow the directions below to ensure you receive the most benefits from this innovative device.

## Instructions:

1. You have already removed the Sleep Noodle™ from its plastic carrying bag. Do not discard this bag as it can be used for storage or when travelling. It will help keep the Sleep Noodle™ clean.
2. You have already chosen your size by following the measuring instructions listed on the outside package before you purchased it. Confirm that you have chosen the correct size by measuring the distance around your chest, just below the underarm. Compare your measurement to the sizing chart on package insert.
3. Wrap the belt around your body around chest level, just below the underarm. If correctly positioned, the black pouch with noodle should be located on back, and the belt on the front. Do not tighten the Sleep Noodle™ belt to the point that you are uncomfortable. It should be loose enough to place two fingers between your chest and belt, while not allowing the belt to rotate around your chest. The inside of the belt is made of a non-slip material to help avoid this rotation. If the belt has been fitted correctly, both the CPAPology™ logo on the front, and the Sleep Noodle™ logo on the back should be right side up.
4. After use, remove the Sleep Noodle by pulling the belts apart. Reattach the two sides of the belt together when not in use; this will reduce the amount of external debris such as hair, lint, and dirt from attaching to the hook & loop fastener, hence reducing its fastening strength.

## Cleaning:

1. Always remove the internal foam noodle before washing the Sleep Noodle™. This is done by unzipping the pouch, and pulling the noodle outwards, past the zipper. The Sleep Noodle™ includes a self-healing zipper. If the zipper separates, simply zip and re-zip to repair. You may be required to bend the foam slightly to pass the zippered area. Never force the zipper open or closed. Always store foam noodle in a clean dry area while cleaning belt.
2. Do not place in washing machine. Hand washing will extend the life of the Sleep Noodle™ and keep hook & loop fastener from picking up material from adjacent clothes. Prepare a warm bath in a sink or container, with a mild detergent. Do not use any type of bleach to clean. Immerse the Sleep Noodle™ belt into the bath for a short period of time. Scrub areas with a soft brush in areas requiring additional attention. Once clean, remove Sleep Noodle™ from bath, and allow to drain excess water. Replace the soapy bath water with clean fresh water. Immerse the Sleep Noodle™ once again for a short period of time, allowing any soap residue to be removed. Remove Sleep Noodle™ from water, allowing time to air dry before reinserting the foam noodle and closing the pouch with the zipper. Do not iron or place in dryer.

## Warranty:

The Sleep Noodle™ includes a 1 year limited manufacturer warranty from time of purchase. In the case of a warranty issue, return the entire Sleep Noodle™ to the location of purchase with the original receipt. At the manufacturer's option, the belt will be replaced or repaired. Please visit [www.CPAPology.com](http://www.CPAPology.com) for other great CPAP products.

**NOTE:** Sleep Noodle helps you to develop the habit of sleeping on your side. We recommend that you give the Sleep Noodle a try for a minimum of 2 weeks.

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Manufactured by KEGO Corporation